RED LAY

RESINA ACRÍLICA

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ACRYLIC RESIN

Technical Datasheet English

Used by both the prosthetist and the dentist, it has low flow and zero contraction, which allows intra-root molding to make molten cores, sculptures of fixed prostheses and the fixation of their welding points, as well as making caskets for molding, fixation of implant transfers and all work with precision is required. Laboratory use.

Composition:

Powder: Polymethyl Methacrylate, benzoyl peroxide, biocompatible pigments.

Liquid: Methyl methacrylate, DMT and Inhibitor.

Precautions:

- * Read the instructions described on the label before handling.
- * For dental use only.
- * Liquid highly flammable, non-toxic (flash point: 10°C / 50°).
- * Powder/ Air mixture (resin) becomes flammable.
- * It is recommended handling in ventilated, nitrile gloves and goggles.
- * Pregnant women, Infants and people sensitive to the product should not handle it.
- * Keep away from light and heat.
- * Do not allow uncured material get in contact with the mucosa of the patient. The heat of the polymerization can cause burns.

* **Colors / Powder:** Red

* **Packaging:** Powder: bottles 25g

* **Liquid:** 30 ml

SHAKE THE BOTTLE BEFORE USING

Instruction for Use:

To perform the molding of intraradicular channel will be important to make a pin guide made with Red lay. Insulate the channel with the insulation (included with the kit), place the powder and liquid into a Dappen pot (indirect technique), with the brush, fill out the channel with resin wetting the brush in the liquid then in the powder and introduced the pin guide.

For making sculpture and skullcaps, isolate the plaster model with Isotins insulator and use the indirect technique as guide, above or direct. Allow a polymerization time of 5 min.

Direct technique:

- * Place the liquid in a Dappen pot and then the powder in the ratio 2: 1.
- * Tool(spatula) the least as possible, cover up the pot Dappen until the plastic phase.
- * Upon reaching the plastic phase resin may be taken to the plaster model was isolated with Isotins.
- At the mouth of the patient.

RED LAY

ACRYLIC RESIN

Technical Datasheet
English

1. Composition and information about ingredients

Ingredients	CAS number	Concentration	Risk Symbol
Polymethyl methacrylate	9011-14-7	N / A	N / A

2. Technical Responsible.

Regiane Rodrigues Nascimento - CRQ 04163305

REGISTRO ANVISA: CLASS: LABORATORY USE

3. Other information

N / A Not applicable N / C Unclassified N / A Not available

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