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LUIZ CARLOS MARCONI– CRQ 04207054

REGISTRO ANVISA: CLASSE: II 80984220003

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FRANTINS INDUSTRY AND TRADE OF DENTAL PRODUCTS LTDA-EPP
BRAZIL – SÃO PAULO – PIRASSUNUNGA – CURITIBA STREET, 4721 – V. BELMIRO/ ZIP CODE 13633-515
PHONE: (19) 3097-1209

Acrylic Resin Polymerizable Bivalent (Powder/Liquid)

For Confection of crowns, bridges, provisional sculptures, facets fixation and jaquets, fillings of retentions in PTR (Prosthesis total removable), PPR (Prosthesis partially removable) and fillings of cavities at works for characterization of artificial acrylic teeth.

Tipe 1; Class 1.

PRESENTATIONS

Packaging:

Powder: Bottles of 80grs.

Liquid Selfpolymerizable with crosslink and fluorescent: Bottles of 50ml and 120 ml.

Liquid thermopolymerizable with crosslink and fluorescent: Bottles of 50 ml.

Colors/Powder: 60, 62, 65, 66, 67, 69 and incisal

Precautions:

- *Read the handling instructions described on the label before using it.
- * For dental use only
- *Flammable liquid, non-toxic material (Flash Point: 10°C / 50°C).
- *Air/Powder mixture (Resin) can form flammable.
- *Handling the product is recommended in ventilated areas, wearing nitrile gloves and protective goggles.
- * Pregnant women, nursing mothers and people with known allergies should not handle this product.
- *Keep protected from heat and light.
- *Do not let a NON polymerized material get on the patient's skin. The heat from polymerization can cause burns.

Composition:

Resin (Powder):

- Polymethyl methacrylate
- Benzoyl Peroxide
- Biocompatible pigments.

Liquid FRANTINS COR Thermopolymerizable:

- Monomer Methylmetacrylate
- EDMA (crosslink)
- Fluorescent Pigments

Liquid FRANTINS COR Selfpolymerizable:

- Monomer Methylmetacrylate
- DMT
- Inhibidor
- EDMA (Crosslink)
- Fluorescent Pigments

Precautions:

- Read the handling instructions described on the label before using it.
- For dental use only
- Flammable liquid, non-toxic material (Flash Point: 10°C / 50°C).
- Air/Powder mixture (Resin) can form flammable.
- Handling the product is recommended in ventilated areas, wearing nitrile gloves and protective goggles.
- Pregnant women, nursing mothers and people with known allergies should not handle this product.
- Keep protected from heat and light.
- Do not let a NON polymerized material get on the patient's skin. The heat from polymerization can cause burns.

SHAKE THE POWDER BEFORE USING IT.

•Preparation:

Containers and hands should be cleaned free of contaminants to prevent changes in color.

For best results always use liquid FRANTINS COLOR with Fluorescent.

Always watch for color and results closer to the scale, attention to observe the correct ratio between the body and the incisal, to lighten the hue of the crown, the layer of the incisal (enamel) is extended along the gingival third. To darken, limit layer to the middle third, exposing more the body color. To achieve intermediate colors, seek mixing proportional shares of two post of the closest to the desired color. For incisal shade darker, adding a quantity of powder of desired color.

Start finishing with drills and cutters pass for abrasive stones and finish it with fine sandpaper texture. The polishing is performed with soft brushes. To ensure the best quality in their work, always make use of monomers FRANTINS COLOR with Crosslink and Fluorescent are specific formulation for this job.

1.Composition and Informations about the raw materials.

Ingredients	CAS Number	Risk Symbols	Concentration
Polymethylmethacrylate	9011-14-7	N/A	N/A

2. Technical Responsible.

- **LUIZ CARLOS MARCONI** – CRQ 04207054

- **Anvisa Registration:** Class: II **80984220003**

3. Other informations.

N/A Not Aplicable N/C Not Classified N/D Not Disponíble.

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