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FRANTINS INDUSTRY AND TRADE OF DENTAL PRODUCTS LTDA-EPP
BRAZIL – SÃO PAULO – PIRASSUNUNGA – CURITIBA STREET, 4721 – V. BELMIRO/ ZIP CODE 13633-515
PHONE: (19) 3097-1209

Indications:

Acrylic Resin and acrylic liquid self-polymerizable, specially developed for orthodontics works.

Type 1; Class 1

Composition:

Powder: Polymethacrylate, Benzoyl Peroxide, Biocompatible Pigments.

Liquid: No Crosslink: Methylmetacrilate and Inhibitor.

Liquid: With Crosslink: Methylmetacrilate, EDMA (crosslink) and Inhibitor.

Precautions:

- Read the handling instructions described on the label before using it.
- For dental use only
- Flammable liquid, non-toxic material (Flash Point: 10°C / 50°C).
- Handling the product is recommended in ventilated areas, wearing nitrile gloves and protective goggles.
- Pregnant women, nursing mothers and people with known allergies should not handle this product.
- Keep protected from heat and light.
- Do not let a NON polymerized material get on the patient's skin. The heat from polymerization can cause burns.

Presentation

Ortodon/COLORLESS Powder: Pots or Refills: 80g, 225g, 450g, 1,000g and 5,000g.

Ortodon Concentrate, 225g Pot, Colors: Blue, Green, Yellow, Red, Black and White.

*Carnival Colors (Glitter): 225g- Colors: Blue, Green, Red, Brown, Violet and Silver- **Upon Request**

Fluorescent Colors: Pot and Refill 80g, 225g, 450g, 1000g, Colors: Blue, Green, Yellow, Orange, Pink, Violet.

*Olympic Metal Concentrate, Pot 225g, Colors: Pearl, Silver, Golden, Bronze, Copper, Gold.- **Upon Request**

Liquid: Refill Bottles (Special Polyethylene) or Glass (Crosslink/ Without Crosslink): 60ml, 120ml, 250ml, 500ml, 1,000ml and Gallon (polyethylene) 5,000ml.

*Ortodon Fluorescent Liquid, Glass, Without Crosslink: 60ml, 120ml, 250ml, 500ml, 1000ml, Colors: Yellow, Pink, Blue, Green, Orange and Violet- **Upon Request**

SHAKE THE POWDER BEFORE USING IT.

Preparation:

All the basic and essential requirements to work with orthodontics must be respected.

Before starting any task, the parts to be used must be completely clean and free of any traces of wax or fat, so the chemical union between the resins may be perfect.

On the indicated applications for the product, it is important a good hydration of the model and an appropriate insulation.

In the powder/liquid technique an amount of powder is placed on the model and saturated with dripping liquid. This way many layers are applied until it reaches the adequate thickness.

The ORTODON resin has a minimum drainage because of that the work can be done faster.

After the application, remove the excess of material and still in the plastic phase, eliminate bubbles through a hydrostatic pressure in the polymerize (place the amount of water needed, apply 20 pounds of compressed air and allow 20 minutes for the polymerization to be completed). After the polymerization is done, start the finishing with drill bit and milling cutter, then with abrasive stones and finalize with fine texture sandpaper. To polish use soft brush and abrasive stone(pumice).

Important

Do not damage wires and clips existent in the device.

ORTODON resin is specifically developed in many colors to increase its area of application by professionals in the field and to stimulate its use by patients. Biocompatible pigments are highly stable. The resins can be mixed to achieve new colors with high stability and abrasion resistant.

1.Composition and Information about the raw materials.

Ingredients	CAS Number	Risk Symbols	Concentration
Polymethylmethacrylate	9011-14-7	N/A	N/A

2.Technical Responsible.

- **Luís Carlos Marconi** – CRQ 04207054

- **Anvisa Registration:** Class: II 80984220001

3.Other information.

N/A Not Applicable N/C Not Classified N/D Not Disponible.

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