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Role = Student

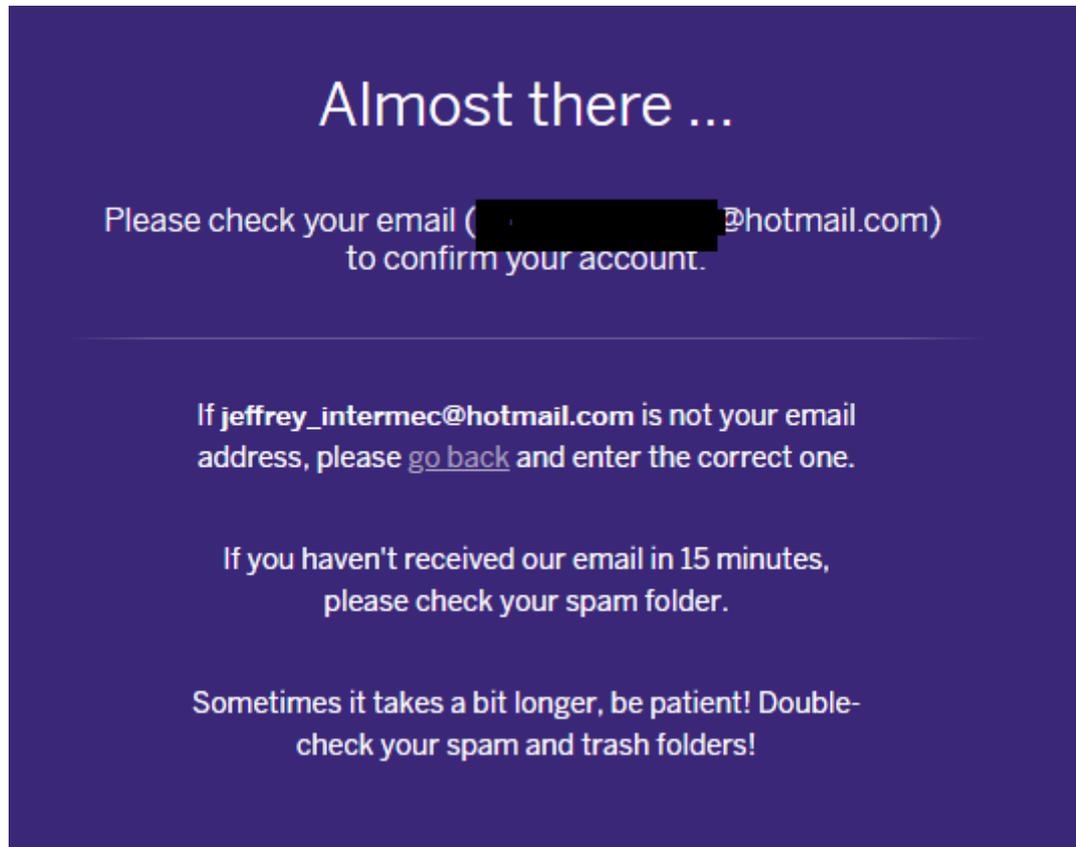
Country = Brazil

Primary development language = Node.js

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Thanks for signing up with Heroku! You must follow this link to activate your account:

<https://id.heroku.com/account/accept/6247335/0718c149d00f96cfb72d196ea71eeb8e>

Have fun, and don't hesitate to contact us with your feedback.

The Heroku Team

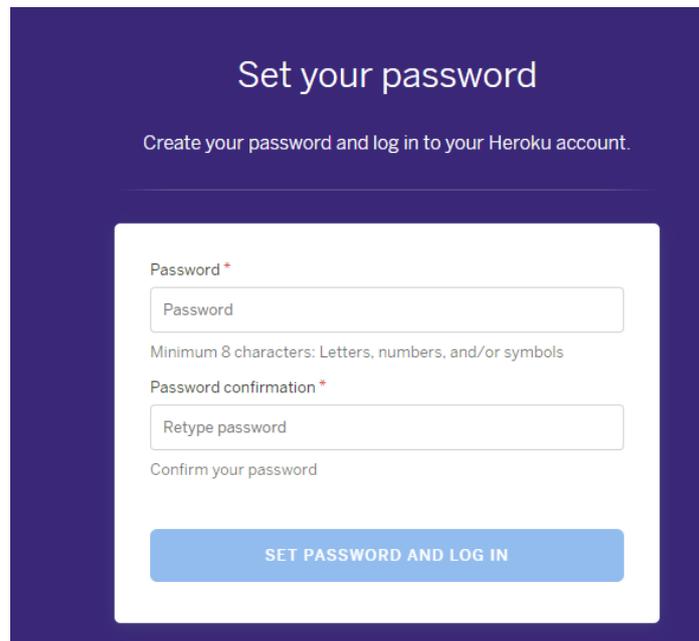
<https://heroku.com>

Heroku is the cloud platform for rapid deployment and scaling of web applications. Get up and running in minutes, then deploy instantly via Git.

To learn more about Heroku and all its features, check out the Dev Center.

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
The screenshot shows a dark purple background with the title "Set your password" in white. Below the title is the instruction "Create your password and log in to your Heroku account." The form itself is white and contains two input fields: "Password" and "Password confirmation". The "Password" field has a red asterisk and a placeholder "Password". Below it is the text "Minimum 8 characters: Letters, numbers, and/or symbols". The "Password confirmation" field has a red asterisk and a placeholder "Retype password". Below it is the text "Confirm your password". At the bottom of the form is a blue button with the text "SET PASSWORD AND LOG IN".

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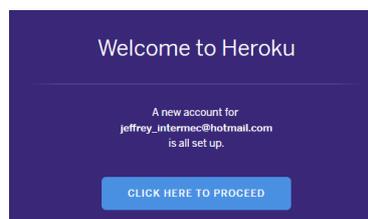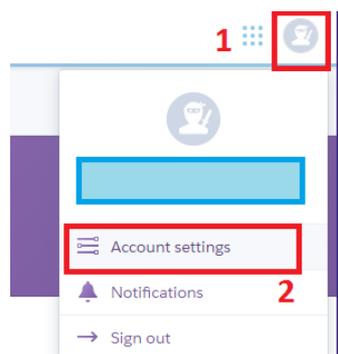
Account Applications **Billing** 1

Billing Information

This account does not have a credit card. **Add credit card** 2

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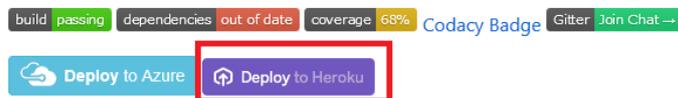
Nightscout Web Monitor (a.k.a. cgm-remote-monitor)

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API_SECRET Required

REQUIRED: A secret passphrase that must be at least 12 characters long, required to enable POST and PUT; also required for the Care Portal.

senhavaiaqui

ENABLE = careportal basal

ENABLE

Used to enable optional features, expects a space delimited list, such as: careportal rawbg iob, see <https://github.com/nightscout/cgm-remote-monitor/blob/master/README.md> for more info.

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A purple rectangular button with rounded corners containing the text "Deploy app" in white.

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